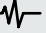
QUESTIONS FRÉQUENTES

MONITEUR

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TEL. +34 965 554 708 - www.practiman.com - international@practiman.com

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CHEST
RECOIL

MEETS
AHA/ERC
FEEDBACK
REQUIREMENTS

PRACTI-BABY PLUS
ELECTRONIC CPR TRAINING

The screen displays two sections: 'Compression rate' and 'Compression depth'. Each section has a score bar (100%, 80%, 60%, 40%, 20%) and a visual indicator (green bar) showing the current performance level. The 'Compression rate' section also includes a heart rate icon and a 'Hi'/'Lo' range indicator. The 'Compression depth' section includes a chest recoil icon and a 'Hi'/'Lo' range indicator.

Buttons include a power button, a directional pad (left, right, up, down), and a monitor button. Labels 'Torso' and 'Monitor' are visible next to battery icons.

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FEEDBACK DE COMPRESSION

FREQUENCE: 100 <-> 120 bpm

Compression rate feedback screen showing a heart rate icon, a green bar, and a green checkmark. Labels 'Hi' and 'Lo' are present.

Compression rate feedback screen showing a red bar and a red 'X' mark, indicating an incorrect rate.

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Compression depth feedback screen showing a chest recoil icon, a green bar, and a green checkmark. Labels 'Hi' and 'Lo' are present.

Compression depth feedback screen showing three examples of incorrect depth and recoil, each marked with a red 'X'.

Score screen showing a green bar at 100% and a green checkmark.

TEST REPORT
⌚
ATTENDRE À 10 SECONDES

Score screen showing a red bar at 0% and a red 'X' mark.

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MODE STANDARD

Mode selection screen showing 'Standard' (eye icon) and 'Blind' (blindfold icon) options with a directional pad.

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Mode selection screen showing 'Standard' (eye icon) and 'Blind' (blindfold icon) options with a directional pad.

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